

INSTRUCTIONS FOR USE

Disposable sampling brush

【Product Name】 Disposable sampling brush

【Product Model】

8101A01;8101A02;8101A03;8101A04;8101A05;8101A06;8101A10
;8101A11;8101A12;8101A20;8101A21;8101A30;8102A01;8102A0
2;8102A03;8102A04;8102A05;8102A06;8110A10;8110A11;8110A
12;8110A40;8110A50;8110A60;8111A10;8111A11;8111A12;8111A
40;8111A50;8111A60;8112A10;8112A11;8112A12;8113A10;8113A
11;8113A12;8112A40;8112A50;8112A60;8113A40;8113A50;8113A
60;8120A01;8120A02;8120A03;8120A10;8120A11;8120A12;8120
A40;8120A50;8120A60;8121A01;8121A02;8121A03;8121A10;812
1A11;8121A12;8130A20;8130A21;8101B01;8101B02;8101B03;81
01B04;8101B05;8101B06;8101B10;8101B11;8101B12;8101B20;81
01B21;8101B30;8102B01;8102B02;8102B03;8102B04;8102B05;81
02B06;8110B10;8110B11;8110B12;8110B40;8110B50;8110B60;81
11B10;8111B11;8111B12;8111B40;8111B50;8111B60;8112B10;811
2B11;8112B12;8113B10;8113B11;8113B12;8112B40;8112B50;811
2B60;8113B40;8113B50;8113B60;8121B01;8121B02;8121B03;812
1B10;8121B11;8121B12;8130B20;8130B21

【Product structure】

Disposable sampling brush consists of a sampling head (with or without a ball tip) and a sampling handle. The sampling head is

stainless steel 304 wound, the bristles are made of PA66 nylon wire, the ball tip is made of epoxy material; the sampling handle is made of PP (Polypropylene).

The Type 8130A20, 8130B20, 8130A21, 8130B21 also contains a sheath.

【Intended Use】

It is mainly used for clinical use of natural channels for biological samples.

【Contraindication】 Non

【Steps to use】

1. Open the pouch, Take out the Disposable sampling brush.
2. Insert sampling head into the sampling site. Apply gentle pressure until the two wing bristles are in close contact with the sampling site.
3. Hold the sampling handle with your thumb and forefinger.

Maintaining gentle pressure, after rotating 2~5 turns in one direction, take out the Disposable sampling brush for sample testing.

【Precaution】

1. Please check the pouch package and the product carefully before use. If either of them is damaged, it is strictly forbidden to use.
2. Please read the instruction manual carefully, pay attention to the expiry date of the product before use. Do not use it after the deadline.

3. This product is disposable. It should be strictly in accordance with the aseptic operation specifications. Please destroy it immediately after use. It is forbidden to reuse.

4. The disposal of wastes shall be carried out in accordance with national environmental protection laws and regulations.

【Definitions of Signs】

	Sterilized using ethylene oxide		Do not re-use
	Do not use if package is damaged		Do not resterilize
	Caution		Fragile, handle with care
	Upward		Keep dry
	Batch code		Use-by date
	Manufacturer		Authorized representative in the European Community
	Date of manufacture		Consult instructions for use
	Temperature limit		Keep away from sunlight
	Stacking limit by number		Medical device
	" CE " Marking		Unique Device Identification

【Storage and transportation】

1. The product should be stored in a room with a temperature of $-10^{\circ}\text{C} \sim 40^{\circ}\text{C}$, relative humidity of not more than 80%, good ventilation, no corrosive gas.

2. The product should be protected from heavy pressure, strong vibration, direct sunlight, high temperature, moisture, rain and corrosive gases or liquids during transportation.

【Sterilization】 EO sterilization

【Shelf Life】 5 years after sterilization



Name: Jiangsu HanHeng Medical Technology Co., Ltd.

Add : #118 Xinyuan 3rd Road, Xinbei District, Changzhou, 213031 Jiangsu P.R. China

Tel: 0086-519-85559288

Web: www.hanhengmed.com



Name: Luxus Lebenswelt GmbH

Add: Kochstr. 1, 47877, Willich, Germany

Contact Person: Liu Sun

Tel/Fax: 0049-1715605732

E-mail: Info.m@luxuslw.de

【Version】 2025.02.14

B/0

