

INSTRUCTIONS FOR USE

Gynecological Kit

【Product Name】 Gynecological Kit

【Product Model】

HH850401, HH850402, HH850403, HH850404, HH850405,
HH850406, HH850407, HH850408, HH850409, HH850410,
HH850411, HH850412, HH850413, HH850414, HH850415,
HH850416, HH850417, HH850418, HH850419, HH850420,
HH850421, HH850422, HH850423, HH850424, HH850425,
HH850426, HH850427, HH850428, HH850429, HH850430,
HH850431, HH850432, HH850433, HH850434, HH850435,
HH850436, HH850437, HH850438, HH850439, HH850440,
HH850441, HH850442, HH850443, HH850444, HH850445,
HH850446, HH850447, HH850448, HH850449, HH850450,
HH850451, HH850452, HH850453, HH850454, HH850455,
HH850456, HH850457, HH850458, HH850459, HH850460,
HH850461, HH850462, HH850463, HH850464, HH850465,
HH850466, HH850467, HH850468, HH850469, HH850470,
HH850471, HH850472

【Product structure】

Gynecological Kit consists of a disposable vaginal dilator, a disposable cervical sampler, a disposable medical drape and disposable examination gloves.

Accessory name	specification	Number
Disposable vaginal dilator	L, M, S	1 pc
Disposable cervical sampler	A, B, C, D	1 pc

Disposable medical drape	50*50, 40*60	1 pc
Disposable examination glove	L, M, S	1 pair

【Intended Use】

It is mainly used for gynecological examination.

【Contraindication】

- 1) Disabled within 2 days before and after menstruation.
- 2) pregnant women are banned.
- 3) Vaginal bleeding is prohibited.

【Steps to use】





















1. Open the product pouch packaging.
2. Spread the medical drape on the examination bed.
3. The patient lies on the medical drape, takes off his pants, and places his legs on the bracket of the examination bed.
4. Gently insert the dilator into the patient's vagina, and then slowly open.
5. After the vaginal dilator is dilated, insert the head of the cervical sampler into the inner cervix until the two bristles and the cervix are in close contact.
6. Thumb and index finger hold the handle, after rotating 2-5 turns in one direction, pull out the cervical sampler.
7. The sampler head is removed and placed directly in the cell preservation solution for sample delivery.

【Precaution】

1. Please check the pouch packaging and the product carefully before use. If either of them is damaged, it is strictly forbidden to use.

2. Please read the instruction manual carefully, pay attention to the expiry date of the product before use. Do not use it after the deadline.
3. This product is disposable. It should be strictly in accordance with the aseptic operation specifications. Please destroy it immediately after use. It is forbidden to reuse.
4. The disposal of wastes shall be carried out in accordance with national environmental protection laws and regulations.

【Definitions of Signs】

	Sterilized using ethylene oxide		Do not re-use
	Do not use if package is damaged		Do not re-sterilize
	Caution		Fragile, handle with care
	Upward		Keep dry
	Batch code		Use-by date
	Manufacturer		Authorized representative in the European Community
	Date of manufacture		Consult instructions for use
	Temperature limit		Keep away from sunlight
	Stacking limit by number		Medical device
	" CE " Marking		Contains latex



Thrown into the
trash after use



Unique Device
Identification

【Storage and transportation】

1. The product should be stored in a room with a temperature of $-10^{\circ}\text{C} \sim 40^{\circ}\text{C}$, relative humidity of not more than 80%, good ventilation, no corrosive gas.
2. The product should be protected from heavy pressure, strong vibration, direct sunlight, high temperature, moisture, rain and corrosive gases or liquids during transportation.

【Sterilization】 EO sterilization

【Shelf Life】 2 years after sterilization



Name: Jiangsu HanHeng Medical Technology Co.,
Ltd.

Add: #118 Xinyuan 3rd Road, Xinbei District,
Changzhou, 213031 Jiangsu P.R. China

Tel: 0086-519-85559288

Web: www.hanhengmed.com



Name: Luxus Lebenswelt GmbH

Add: Kochstr. 1, 47877, Willich, Germany

Contact Person: Liu Sun

Tel/Fax: 0049-1715605732

E-mail: Info.m@luxuslw.de

【Version】 2025.02.14

B/0

